NITRIILIKÄSINE ETRA 12-300

KÄYTTÖOHJE

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USER MANUAL

Glove name: Nitrile glove Etra 12-300

Description: Thin, disposable nitrile glove, powder-free, 3–4 mil, (palm 0.07 mm)

CE category: Category III

Surface finish: Finger-textured

Glove model: ETRA 12-300

Brand: ETRA

GLOVE DESCRIPTION

100% nitrile, powder-free glove that provides protection against specified low-risk solvents and mild chemicals.

The glove features a rolled cuff that offers additional protection from potential chemical splashes.

It is ambidextrous and provides good tactile sensitivity. Not recommended for mechanical risks.

The materials or processes used in manufacturing are not known to be harmful to the user.

The product complies with the EU Personal Protective Equipment Regulation (EU) 2016/425.

It meets the requirements of REACH Regulation 1907/2006, Annex XVII, item 50, regarding PAH compounds.

Glove pH: 6.0–9.0 and tactile sensitivity level: 5.

EN ISO 374-1:2016+A1:2018/Type B

KPT

EN ISO 374-5:2016

VIRUS

THE EU TYPE-EXAMINATION (MODULE B) HAS BEEN CARRIED OUT BY:

SATRA Technology Europe Ltd
Bracetown Business Park, Clonee,
Dublin, D15 YN2P, Ireland
(Notified Body No. 2777)

PERMEATION OF CHEMICALS IN ACCORDANCE WITH EN ISO 374-1:2016+A1:2018 / TYPE B

CHEMICAL	CODE LETTER	PERMEATION TIME	PERFORMANCE LEVEL
Methanol	A	< 1 minute	0
n-Heptane	J	< 10 minutes	0
Sodium hydroxide 40 %	K	> 480 minutes	6
Sulfuric acid 96 %	L	6 minutes	0
Ammonium hydroxide 25 %	O	< 10 minutes	0
Hydrogen peroxide 30 %	P	31–45 minutes	2
Formaldehyde 37 %	T	361–480 minutes	5

**PERMEATION OF CHEMICALS NOT LISTED IN THE STANDARD
EN ISO 374-1:2016+A1:2018.**

CHEMICAL	CODE LETTER	PERMEATION TIME	PERFORMANCE LEVEL
Hydrochloric acid 10%	-	>480 minutes	6
Isopropanol 70%	-	6 minutes	0
Ammonium hydroxide 1%	-	> 480 minutes	6
Hydrogen peroxide 18%	-	>480 minutes	6

Performance level classification	1	2	3	4	5	6
Permeation time (min)	>10	>30	>60	>120	>240	>480

The lowest Breakthrough time is used determination of performance level.
Level 0 – Performance Level 1 not achieved.

The test results are based on the palm area of the glove. The glove’s performance is based on laboratory data and may not correspond to actual protection time in the workplace, as it can vary depending on mixtures versus pure chemicals and other performance-affecting factors such as temperature, abrasion, and degradation.

When in use, protective gloves may offer reduced protection against hazardous chemicals due to changes in physical properties. Movements, gripping, rubbing, and degradation caused by chemical contact can significantly reduce the actual wearing time.

For corrosive chemicals, degradation may be the most critical factor when selecting chemical-resistant gloves.

DEGRADATION CAUSED BY CHEMICALS IN ACCORDANCE WITH EN ISO 374-4:2019

CHEMICAL	CODE LETTER	AVERAGE DEGRADATION	APPEARANCE AFTER TESTING
Sodium hydroxide 40 %	K	15,1%	Softened
Hydrogen peroxide 30 %	P	-74,7%	Swollen
Formaldehyde 37 %	T	24,0%	Softened
Methanol	A	50,4%	Swollen and softened
n-Heptane	J	29,1%	Hardened
Sulfuric acid 96 %	L	100%	Disintegrated
Ammonium hydroxide 25%	O	30,9%	Softened
Hydrochloric acid 10%	-	23,6%	Softened
Isopropyl alcohol 70%	-	59,5%	Softened
Ammonium hydroxide 1 %	-	34,3%	Softened
Hydrogen peroxide 18 %	-	18,7%	Softened

Degradation levels indicate the change in the puncture resistance of gloves after exposure to the test chemical.

Risk of microorganisms in accordance with EN ISO 374-5:2016

Protection against bacteria and fungi: APPROVED

Protection against viruses: APPROVED

Penetration resistance has been evaluated in the laboratory and applies only to the tested sample.

Markings: Manufacturer's name, performance level pictograms, model, size, CE marking, and the notified body number.

This information is presented on the glove or its primary packaging.

Misuse: Poor fit reduces tactile sensitivity and finger dexterity, which may cause hand and finger fatigue.

Incorrect size and poor fit result in inadequate hand protection.

Donning the glove: Ensure the correct size is selected. Check that the glove fits properly.

Inspect for any damage, condition issues, or contamination before each use.

Glove doffing : Remove the glove immediately if any signs of wear appear. If the glove is contaminated with hazardous substances, remove the gloves one at a time using the opposite gloved hand to avoid contact between the contamination and bare skin.

Cleaning/Maintenance: Check the gloves for defects or damage before use. Do not clean mechanically. For single use only.

Hand hygiene: Hands should be washed before putting on gloves and after removing them.

Storage: Store in a dry place at a temperature between 10–30 °C in the original packaging. Do not exceed 40 °C.

Expiration: When stored according to the recommendations, the chemical properties remain intact for up to three years from the date of manufacture. The service life depends on the intended use, and it is the user's responsibility to ensure suitability.

Disposal: Used gloves should be disposed of by incineration or in accordance with local regulations.

Available sizes: XS, S, M, L, XL and XXL

The glove complies with Regulation (EC) No 1935/2004 and is suitable for short-term contact with all types of food.

General information: It is recommended to ensure the suitability of the gloves for the intended use, as workplace conditions may differ from type testing in terms of temperature, abrasion, and degradation.

Warning: The test results are intended to assist in the selection of gloves, but it must be understood that actual conditions of use cannot be fully simulated. The assessment of the glove's suitability for the intended use is the responsibility of the user, not the manufacturer. Further information may be obtained from manufacturer.

The Module D assessment was carried out by: SGS Fimko Oy, Takomotie 8, FI-00380 Helsinki, Finland. (Notified Body No. 0598). See the attached EU Declaration of Conformity.

Declaration of conformity and user manual available for download at www.etra.fi.

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Mooduli D hindamise viis läbi: SGS Fimko Oy, Takomotie 8, FI-00380 Helsingi, Soome. (Teavitatud asutus nr 0598).